



Ultimate Adjustable Threshold Ramp

The legs of the ramp can be setup two different ways:



A) The black adjusting knob can be setup to cover the bottom nut allowing it to be used to turn the legs.

B) The black adjusting knob can be setup to cover the top nut allowing to move the nut up or down.

Installing the legs on your adjustable threshold ramp:



fig. 1

1) Insert the threaded leg into the hole on the underside of the ramp twist till desired height.

2) Once need the legs have been twisted to correct height add (fig.2) twist on the knob on the top of the leg (fig. 3)



fig. 2



fig. 3

Permanent mounting instructions (optional):

1) Drill (2) 1/4" holes in the bottom of the ramp.

2) Bolt threshold down with proper screws for the floor surface.

